


RAC Wireless Contact

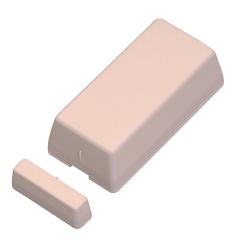

- Easy to use and compact
- Battery life-time of 3 years
- Automatic radio daily message
- Low battery monitoring

The RAC Wireless Contact detects the opening or closing of an electrical contact or of a door equipped with a magnet. This will activate a radio signal to a NurseCall unit. The RAC Wireless Contact provides magnetic reed contacts for monitoring doors and windows. The internal reed contacts are activated by an external magnet which can be placed on the marked position. It also has a built-in screw terminal as input for monitoring external alarm device.

Functions

To use as a transmitter for particular events, the alarm can be activated by approaching and/or moving away a magnet, or by the opening and/or closing of an electrical contact. In this case, the alarm is immediately transmitted to the NurseCall system.

Reed Contacts

There is a reed contact on the front side of the device, as in the picture, as well as the left side of the device. The magnet assembly can be mounted within a certain distance of the device in front of the chosen position. When the distance between the magnet and the internal reed switch of the RAC Wireless Contact exceeds a pre-defined value, a radio signal is sent to the NurseCall unit.

Use as a Universal Point Transceiver

For use as a universal point transceiver, an external device can be connected to the terminal block of the RAC Wireless Contact. A closing or opening signal from the external device initiates a radio signal from the RAC Wireless Contact to the NurseCall unit.

Certifications and Approvals

R&TTE 1999/5/EC EMC 2004/108/EC LVD 2006/95/EC EN 50371 (2002) EN 55022 (2006, A1: 2007) EN 60950-1 (2011) EN 300220-2 V2.3.1 EN 301489-1 V1.8.1 EN 301489-3 V1.4.1 EN 50130-4 (1995, A1: 1998, A2: 2003)

Parts Included

| Q | uantity | |
|---|---------|--|
| 1 | - | |

Component RAC Wireless Contact

Installation/Configuration Notes

Recommendations:

- Allow a minimum height of 20 cm from the ground.
- Do not use on metallic surfaces. If this is not possible, use spacing plates.
- Install the magnet on the moving part (door or window) and the radio transmitter on the frame.
- Avoid using "double side" adhesive tape.
- The RAC is not waterproof.

Technical Specifications

| Dimensions (H x W x D) | 85 x 40 x 40 mm |
|--------------------------------------|---|
| Weight | 55 g |
| Frequency | 434.01 MHz |
| Protection class | IP20 |
| Power supply | 3V lithium battery CR2032 |
| Consumption | Transmission: < 16 mA Idle mode: < 3 µA |
| Battery life | 3 years, under the following conditions: 3 alarms transmissions without repetition per day and 1 daily test transmission (incl. battery status) |
| Battery alarm | at approx. 20 % remaining |
| Automatic battery test | daily check and at every alarm |
| Permissible temperature | 0 °C to 55 °C |
| Frequency stability (0 to +55 °C) | < ± 2 KHz |
| Modulation | FSK |
| Antenna | Integrated |
| Radiated power | 2 mW |
| Transmission time | 2 x 1300 ms |
| Pause between transmissions | 200 to 600 ms (random) |
| Minimum range in free field | 250 m |

TeleAlarm SA Rue du Pont 23 2300 La Chaux-de-Fonds Switzerland info@telealarm.com www.telealarm.com **TeleAlarm UK Ltd.** 3 Bedford Place

WC1B 5AH, London United Kingdom info-uk@telealarm.com www.telealarm.com