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Safety notes

The Vibby OAK should always be worn on the wrist.

Repairs and battery change should be carried out by authorized personnel only.

When a "battery low" warning is transmitted by the Carephone, the battery must be replaced as soon as possible. As a preventive measure, the battery must be replaced after 24 months of operation.

Activate the radio transmission monitoring in your Carephone. Tests confirming the correct functioning of the Vibby OAK must be carried out regularly.

Starting / Exit from sleep mode

Press and hold the button for more than 3 seconds.

Release the button while the LED blinks.

The LED turns off.

The Vibby OAK enters the demonstration mode for 5 minutes (confirmed by the flashing green LED). This mode, used by authorized personnel only, allows the simulation of falls. Then, Vibby OAK switches to active mode.

How do I register the Vibby OAK into the Carephone?

Set the Carephone in programming mode (see Carephone user manual).

The Vibby OAK must be registered as two separate transmitters (two different radio codes).

Register the emergency call function

Press and release the button.

The LED lights up once.

The Carephone stores the radio code.

Register the fall detection function

Press and hold the button.

The LED lights up for 3 seconds, then blinks twice and turns off.

The button can then be released.

After registering, test the function**Trigger an emergency call**

Upon short press (less than 3 seconds) the LED lights up.

The detector then sends its alarm signal.

Automatic fall detection

User activity (walking) followed by a loss of balance leading to a heavy/dangerous fall without returning to a standing position, as well as a loss of consciousness not allowing the user to activate the push button.

Before sending an automatic alarm to the Carephone, Vibby OAK vibrates and blinks for 20 seconds intermittently. During this period if the user does not wish the alarm to be sent, he should place his hand over the Vibby OAK until the vibrations stop. The sending of the alarm will be cancelled.

The fall detection technology used by the Vibby OAK does not allow analysis and interpretation of all fall situations. Soft falls, descent-controlled falls against a wall or a chair, fall down stairs, etc., may not be detected by the Vibby OAK. Should you have a fall and you do not feel the vibrations from the Vibby OAK, just press the help button on the Vibby OAK to manually raise an alarm call.

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