



Netrix- Suicide check integration



Netrix- Suicide check integration
Klik hier als u tekst wilt invoeren.

11/09/2013

Copyright

This document may not be copied in whole or in part or otherwise reproduced without prior written consent from IndigoCare Europe or IndigoCare USA, except where specifically permitted under international copyright law.

Disclaimer

The information in this document is subject to change without notice. IndigoCare Europe or IndigoCare USA assumes no responsibility for inaccuracies or omissions and specifically disclaims any liabilities, losses, or risks, personal or otherwise, incurred as a consequence, directly or indirectly, of the use or application of any of the contents of this document. For the latest documentation, contact IndigoCare Europe or IndigoCare USA.

This publication may contain examples of screen captures and reports or database used in daily operations. Examples may include fictitious names of individuals and companies. Any similarity to names and addresses of actual businesses or persons is entirely coincidental.

All examples of patient information in this document are fictitious. Any resemblance to a real patient or facility is purely coincidental. The owners and users of this product are solely responsible for complying with all applicable patient information laws. The users, by their use of this product, agree to indemnify the manufacturer and/or seller of this product against all claims, litigation, and suits filed for patient information violations.

Intended use

Use this product only for the purpose it was designed for; refer to the data sheet and user documentation for details. For the latest product information, contact IndigoCare Europe or IndigoCare USA.

This hardware installation guide is a working and supporting tool for certified engineers for installing and maintaining iCall installations.

FCC compliance

This equipment has been tested and found to comply with the limits for a Class B, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications.

You are cautioned that any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Contact information

IndigoCare USA
1001 South Dairy Ashford, Suite 100
Houston, TX 77077
USA

T: +1 713.595.6698
F: +1 713.595.6699
Email: usa@indigocare.com
www.indigocare.com

IndigoCare Europe
Schoebroekstraat 48
3583 Paal
Belgium

T: +32 11 247090
F: +32 11 247099
Email: Europe@indigocare.com
www.indigocare.com

Content


- 1. General purpose5
- 2. Implementation.....5
- 3. Programmation8
- 4. Website integration 10

1. General purpose

This document describes the implementation, programming and testing of the Welfare-Suicide Check interface in the Netrix software. The goal is to monitor a selection of rooms, and automatically warn the appropriate person if there hasn't been an action in a room for a selected amount of minutes.

2. Implementation

To enable the Welfare-Suicide Check interface, go to Input-Output screen via the menu "Setup→Input-Output" and activate the interface (license). Select Suicide check and input the amount of minutes after which an alarm should be triggered if there haven't been any actions for a selected room.

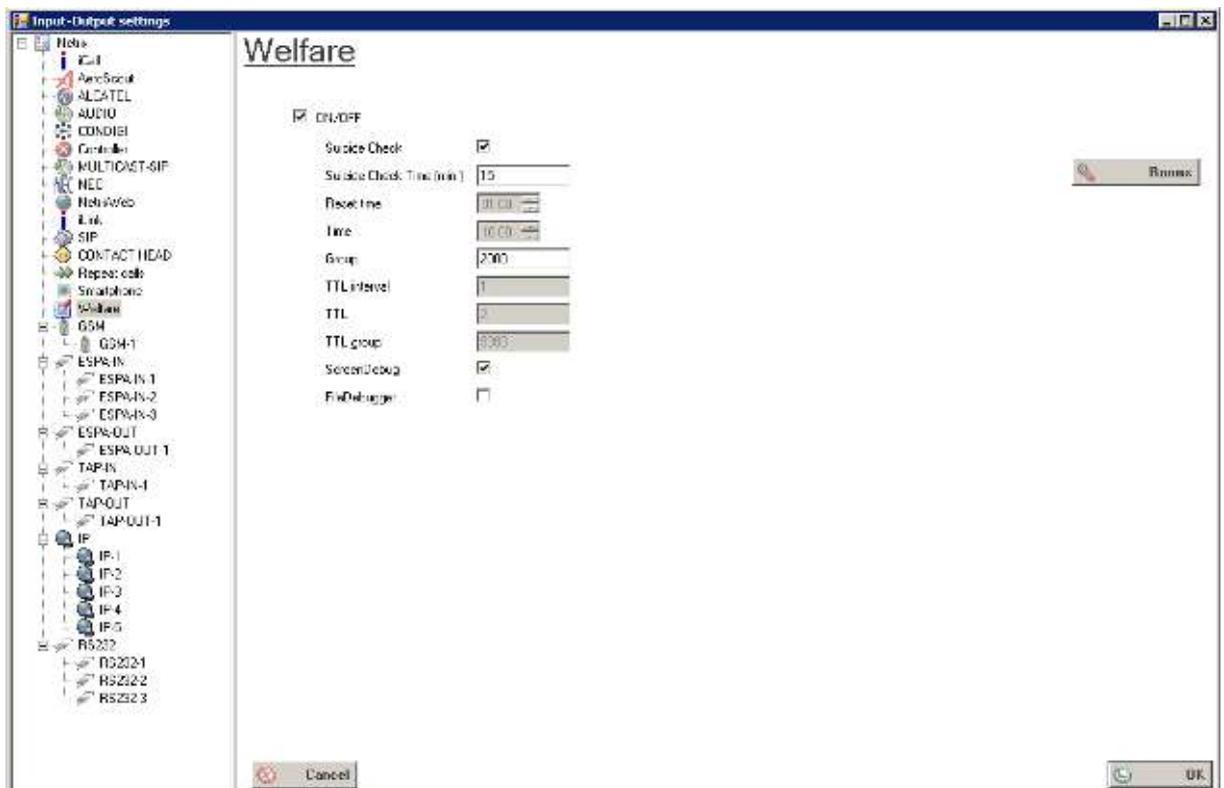

Group: The group number to be called when there hasn't been activity in a room after the "Suicide check Time".


ScreenDebug: check this box if you need a debug screen for this interface

FileDebug: check this box to log the debug info


Errorlogging: Check this box to send errors to the errorlog.

Rooms

Click on the Rooms button to edit the appropriate rooms.

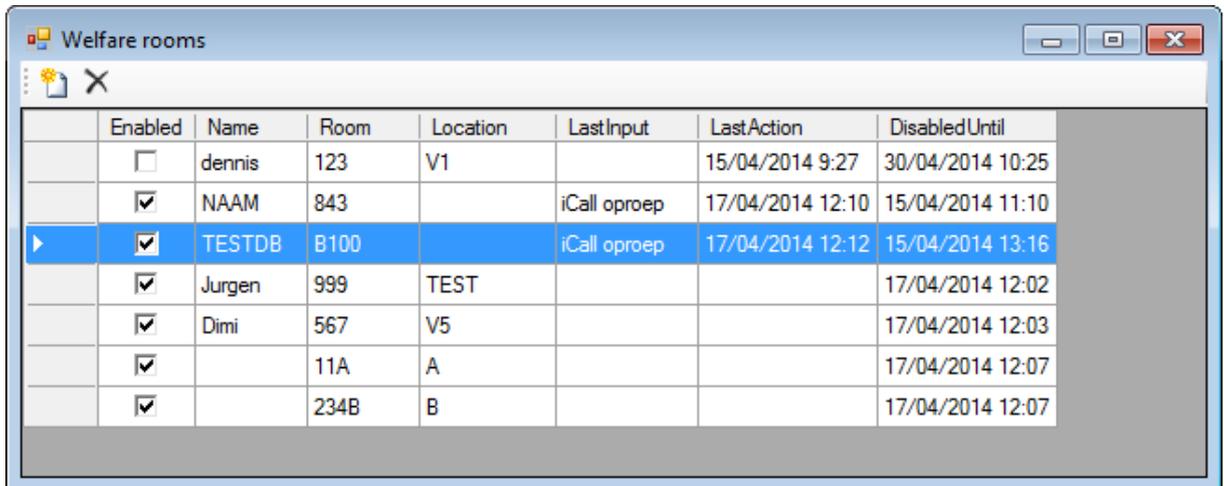

	Enabled	Name	Room	Location	LastInput	LastAction	DisabledUntil
	<input type="checkbox"/>	dennis	123	V1		15/04/2014 9:27	30/04/2014 10:25
	<input checked="" type="checkbox"/>	NAAM	843		iCall oproep	17/04/2014 12:10	15/04/2014 11:10
▶	<input checked="" type="checkbox"/>	TESTDB	B100		iCall oproep	17/04/2014 12:12	15/04/2014 13:16
	<input checked="" type="checkbox"/>	Jurgen	999	TEST			17/04/2014 12:02
	<input checked="" type="checkbox"/>	Dimi	567	V5			17/04/2014 12:03
	<input checked="" type="checkbox"/>		11A	A			17/04/2014 12:07
	<input checked="" type="checkbox"/>		234B	B			17/04/2014 12:07

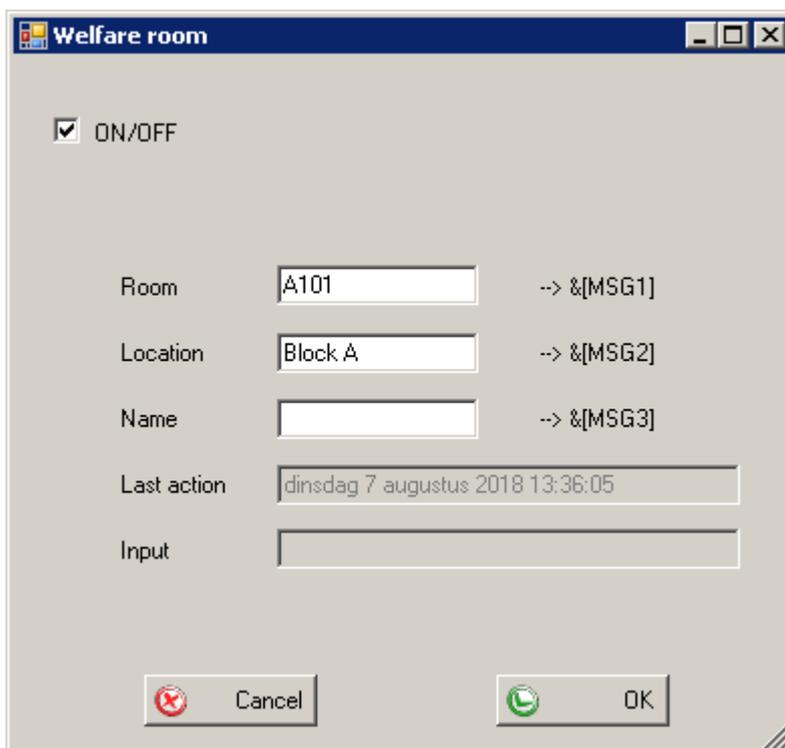

ON/OFF

Room: --> &[MSG1]

Location: --> &[MSG2]

Name: --> &[MSG3]

Last action:

Input:

If a room hasn't been checked for x minutes, a group call will be triggered and room details (Room number, location, name) will be sent to this group.

The current state is displayed in a window in the main screen.

Enabled	Room	Location	Name	LastAction	LastInput
<input checked="" type="checkbox"/>	A101	Block A		7/09/2018 10:51...	iCall Netix
<input checked="" type="checkbox"/>	C102	Block C		7/09/2018 13:51...	iCall Netix
<input checked="" type="checkbox"/>	C105	Block C		7/09/2018 8:30:02	

Count=3

Red: Suicide check active (Message has been sent to group)

Green: Suicide check cancelled

With iCall, it is possible to set the appropriate room in a Suicide check state by broadcasting a Toilet status to this room. This action will change the room status, and hereby change the color in the Welfare screen as well (→ back to green).

Participant

Name:

Type: IP iCall room call

IP: iCall

Room: &[MSG1]

Type of call: 3 - Toilet

Addition:

Location: &[MSG2]


Nurse:

Buttons: Cancel, OK

3. Programmation

To reset a room (register activity in a room), create or select a group and include a welfare message.

Select type “Welfare”.

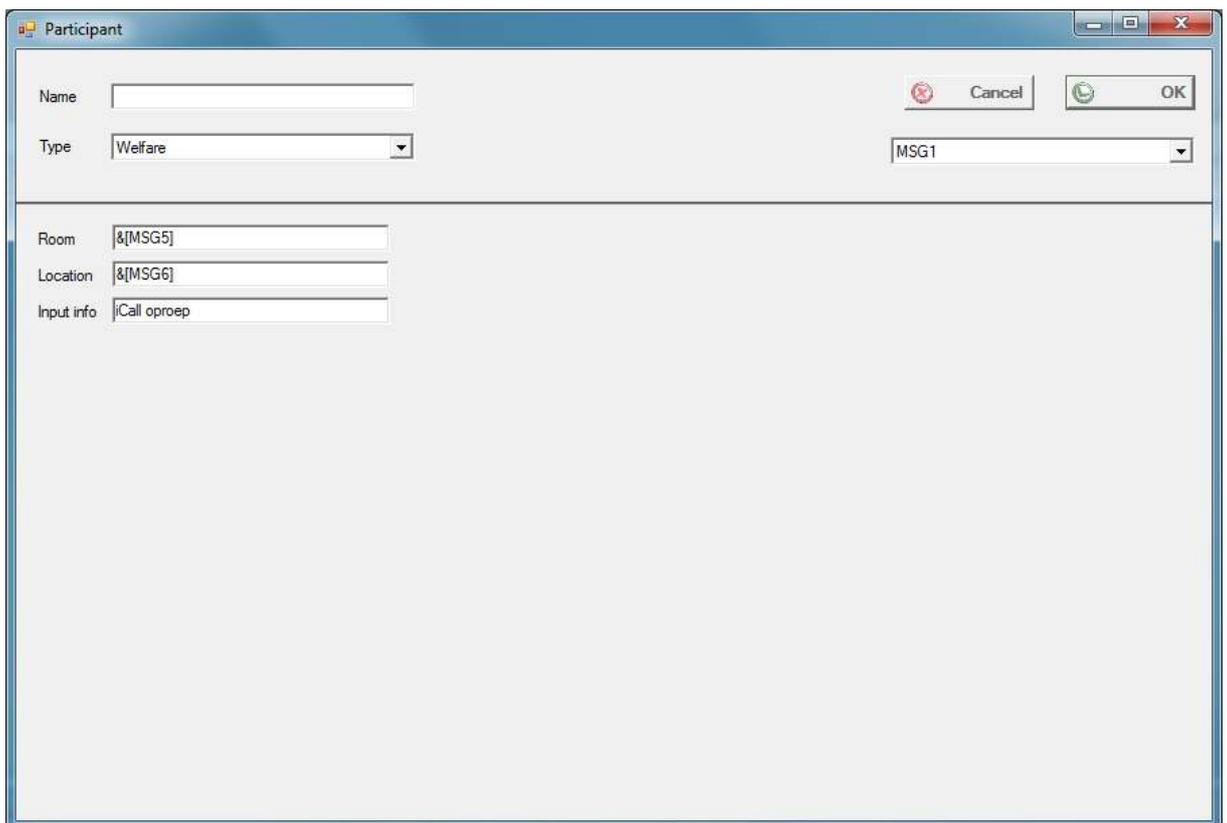

The screenshot shows a dialog box titled "Participant". It has a standard Windows-style title bar with minimize, maximize, and close buttons. The dialog contains the following fields and controls:

- Name:** An empty text input field.
- Type:** A dropdown menu with "Welfare" selected.
- MSG1:** A dropdown menu with "MSG1" selected.
- Room:** A text input field containing the text "&[MSG5]".
- Location:** A text input field containing the text "&[MSG6]".
- Input info:** A text input field containing the text "iCall oproep".
- Buttons:** "Cancel" and "OK" buttons are located in the top right corner.

Room: the Room number

Location: The location of your room (can be left empty).

Room and location should match a room programmed in the Welfare setup (Input-Output)

Input info: additional info, for example to specify the source of the call.

Participant

Name: _____

Type: |

Room: | Location: | iCall group:

Welfare rooms

Enabled	Name	Room	Location	LastInput	LastAction	Dis
<input type="checkbox"/>	denis	123	V1		15/04/2014 9:27	30/0
<input checked="" type="checkbox"/>	NAAM	843		iCall group	17/04/2014 12:10	15/0
<input checked="" type="checkbox"/>	TEST08	E100		iCall group	17/04/2014 12:12	15/0
<input checked="" type="checkbox"/>	Jugen	999	TEST			17/04/2014 12:00
<input checked="" type="checkbox"/>	Dini	967	V5			17/04/2014 12:03
<input checked="" type="checkbox"/>		11A	A			17/04/2014 12:07
<input checked="" type="checkbox"/>		234B	B			17/04/2014 12:07

Count: 7

Welfare

Enabled	Room	Location	Name	LastAction	LastInput
<input checked="" type="checkbox"/>	123	V1	denis	15/04/2014 9:27	
<input checked="" type="checkbox"/>	843		NAAM	17/04/2014 12:10	iCall group
<input checked="" type="checkbox"/>	8100		TEST08	17/04/2014 12:12	iCall group
<input checked="" type="checkbox"/>	999	TEST	Jugen	17/04/2014 12:00	
<input checked="" type="checkbox"/>	967	V5	Dini	17/04/2014 12:03	
<input checked="" type="checkbox"/>	11A	A		17/04/2014 12:07	

4. Website integration

To manage the rooms, also a website is available.

Browse to the Netrix website <http://xxx.xxx.xxx.xxx/Netrix> where xxx.xxx.xxx.xxx stands for the IP address of the Netrix. Log in with the appropriate credentials. If you have a valid license, an additional link “Welfare” will appear in the menu.



You can now manage the rooms.



