Manual iCall MIVA SET V1.0

Manuel du système iCall MIVA SET V1.0

Handbuch iCall MIVA SET V1.0

Handleiding iCall MIVA SET V1.0

Table of contents

English manual.....	2
Manuel du système française	14
Deutsch Handbuch.....	26
Nederlandstalig handboek.....	38

Manual iCall MIVA set V1.0

Copyright

This document may not be copied in whole or in part or otherwise reproduced without prior written consent from IndigoCare Europe or IndigoCare USA, except where specifically permitted under international copyright law.

Disclaimer

The information in this document is subject to change without notice. IndigoCare Europe or IndigoCare USA assumes no responsibility for inaccuracies or omissions and specifically disclaims any liabilities, losses, or risks, personal or otherwise, incurred as a consequence, directly or indirectly, of the use or application of any of the contents of this document. For the latest documentation, contact IndigoCare Europe or IndigoCare USA.

The users, by their use of this product, agree to indemnify the manufacturer and/or seller of this product against all claims, litigation, and suits filed for patient information violations.

Intended use

Use this product only for the purpose it was designed for; refer to the data sheet and user documentation for details. For the latest product information, contact IndigoCare Europe or IndigoCare USA.

You are cautioned that any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Contact information:

For US-Region:

IndigoCare USA
1001 South Dairy Ashford, Suite 100
Houston, TX 77077
USA

T: +1 713.595.6698
F: +1 713.595.6699
@: usa@indigocare.com
www.indigocare.com

For EMEA-Region:

IndigoCare Europe
Schoebroekstraat 48
3583 Paal
Belgium

T: +32 (0) 11 247 090
F: +32 (0) 11 247 099
@: europe@indigocare.com
www.indigocare.com

Contents

1. Introduction 5

2. Features 5

3. Functionality 5

4. Components 6

5. Installation 7

6. Part Numbers 12

7. Pull String installation 13

1. Introduction

The MIVA call system is designed for use in disabled toilets. The system is suitable for various applications where it makes sense to provide signalling for potentially dangerous situations. The alarm tone is local on the room Lamp and can be send to a central staff display as well. A maximum of 8 MIVA sets can be monitored on the central staff display. There is no need for a configuration via PC.

The complete set is delivered in a single box and contains all components (except wiring) that are necessary for the installation of a single toilet space.

2. Features

- Simple and quick installation
- Hygienic & aesthetic
- Standalone set
- Manufactured entirely from rustproof materials

3. Functionality

- LED room Lamp can be connected to multiple call points and operates as central control module
- Multiple Miva sets can be connected to a single central staff display
- Visual and acoustic alarm signal
- Pull String and Call/Cancel unit available as “normally open” or “normally closed” type
- According NEN 1814 standard, Pull String must be installed such that its lower end is between 300 and 450 mm above floor level.
- Set Complies with the NEN 1814 standard
- Reassurance LED

4. Components

Room Lamp

The room lamp is installed outside the toilet room. Upon activation of a call, a red LED will indicate that the person in the toilet room needs help.

Mode 1: LED flashes at a rhythm of 1 sec ON and 1 sec OFF, in addition an acoustic sound is sent every 1 sec to get attention on the call.

Mode 2: LED is stable ON, in addition an acoustic sound is sent every 17 sec to get attention on the call.

Pull String switch

By activating the pull string an alarm is generated, a confirmation LED at the pull string wall box is the confirmation of the call.

This confirmation Lamp will flash at a cadence of 1 second in Mode 1 and will Lamp steady in Mode 2.

Install the pull string between 300 and 450mm above the floor and alongside the toilet walls.
(regulation NEN1814)

Call and Cancel button The call and cancel button must be located in the room. The red button is an extra call button and has a confirmation LED. By pressing the green button the call from the room is cancelled.	
Power supply The power supply is a 24V DC 900mA adaptor.	

5. Installation

5.1. Mode Selection

In the front of the room Lamp is a small selection button. By pressing this button you toggle between Mode 1 and Mode 2. In case of power failure the selected mode will still be active on repowering of the unit.

<u>Mode 1:</u>	LED	1 Sec ON	1 Sec OFF
	Buzzer	1 Sec ON	1 Sec OFF
	Output	1 Sec ON	1 Sec OFF
<u>Mode 2:</u>	LED	Continued ON	-
	Buzzer	1 Sec ON	17 Sec OFF
	Output	Continued ON	-


5.2. Normally open or normally closed connected

The call points can be connected as a normally open or normally closed input. For field safe reasons we advise to use the normally closed solution.


In case of using a normally open connection the second input must be provided with a small wire bridge.

5.3. Wiring installation


Power connection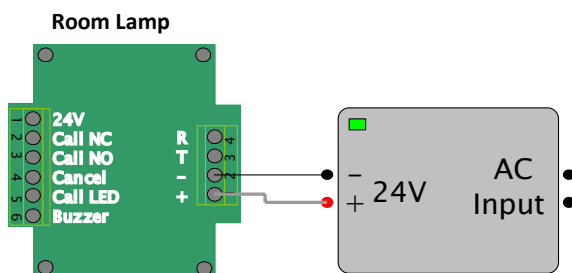


Buzzer connection

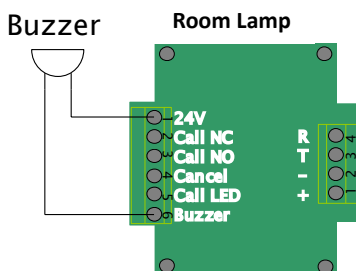
Pull String connection (**normally closed**)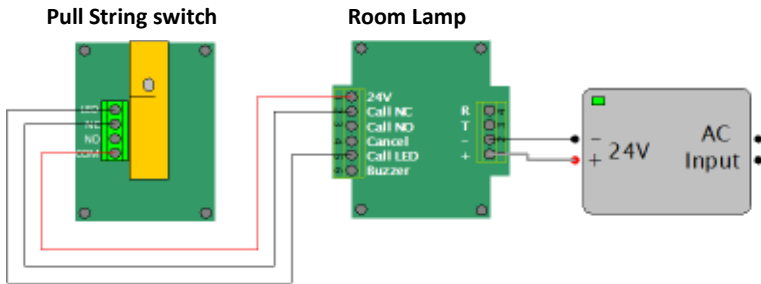


Pull Cord connection (**normally open**)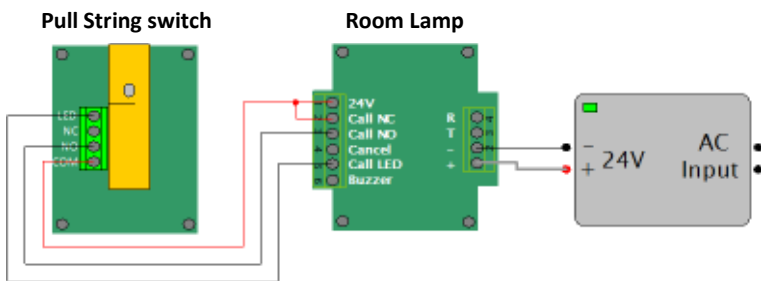

Call and Cancel button (**normally closed**)

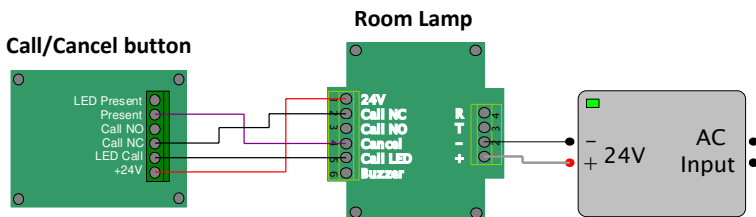
Call and Cancel button (**normally open**)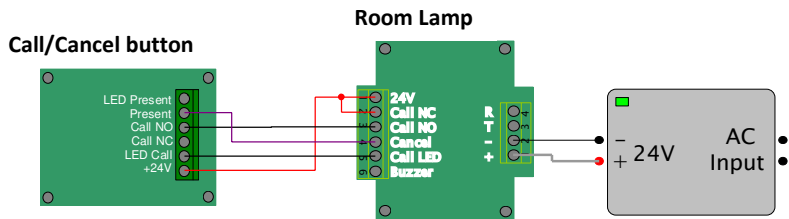

Complete diagram for normally open connection


Call/Cancel button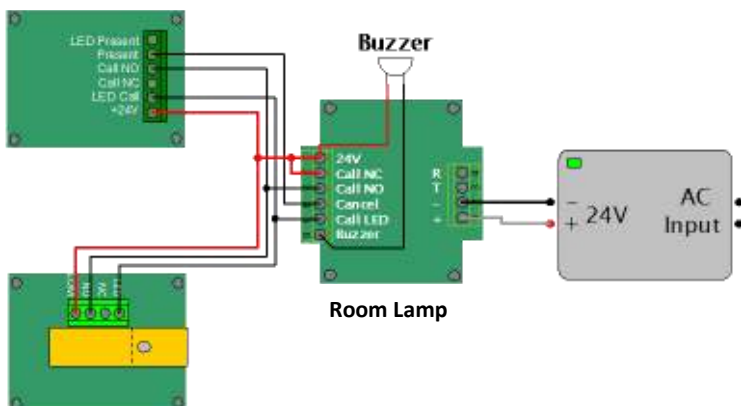

Pull String switch

Complete diagram for normally closed connection

Call/Cancel button

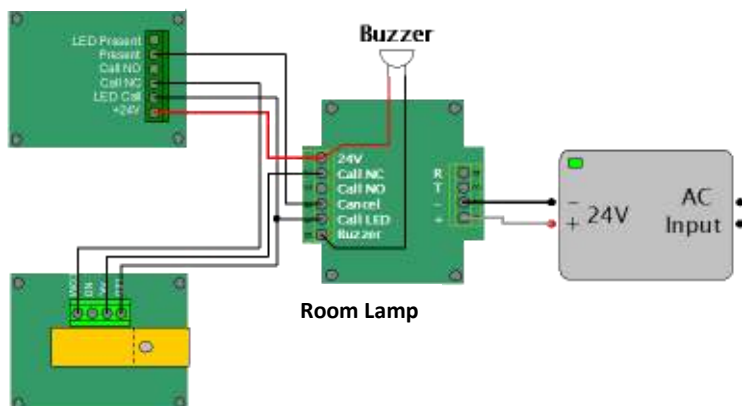Pull String switch

6. Part Numbers


NWBAES5000	Complete MIVA set
NWBAES4500	Pull String button
NWBAES4101	Call and Cancel button
PWBXGA0104	Pull String
NWAABI0100	Power supply
NWAABI0200	AC-Cord
NWBAES3852	Room Lamp

7. Pull String installation

This set includes:

- 4-metre non-stretch red polypropylene cord
- Elastic tensioner with tension releaser and nylon hook
- 4 nylon pulley plugs (incl. push-in plug and ring)
- Cord and button with plug

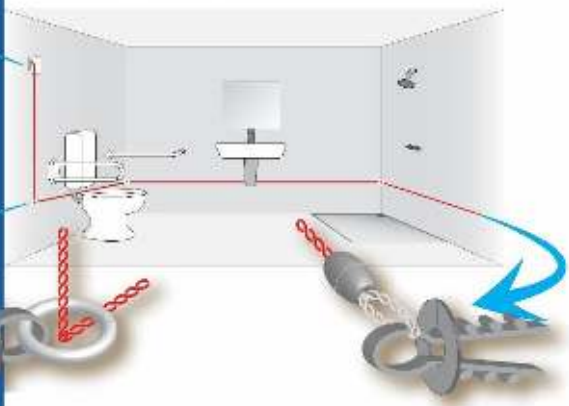Installation:
Install the pull cord between 300 and 450mm above the floor and alongside the toilet walls. Insert the rings into the drill holes, using the plugs. Slide the pull cord through the rings and attach the hook to the pull cord button. Tighten the pull cord, without releasing a call, and fixate it by making a knot behind the last plug.

Drilling advice:

Hard stones: Ø 6,0mm
Soft stones: Ø 5,5mm

- Simple and quick installation
- Hygienic & aesthetic
- Manufactured entirely from rustproof materials

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