

Title	Status	Date	Version
L071TX FW description file	Final	02.06.2023	1.5

1 Forewords

This document applies to the NurseCall L071TX FW programming, dedicated to the N86 product.

Files are supplied in binary format.

2 L071TX firmware:

Software SAP#:	T.040.001.548
Software SAP Description:	Software N86
Software information to be printed	None

2.1 Binary firmware version details:

File format:	binary
Version:	NC_04.A00.05.1502
Build date:	02.06.2023
Compiler version	IAR EWARM 8.50.6
File Name:	NC_04.A00.05.1502.bin
MD5 Checksum:	AD4F8CBC86EFF68E220BBCBF17F43CC4
Remark:	None

2.2 Change History:

SW Version	Build date	Description
04.A00.01.1395	29.06.2021	Ready for Ramp-up production
04.A00.01.1422	01.11.2021	Fixed character problem in Production initialization
04.A00.02.1483	15.11.2022	ADC closing reviewed after Battery measurement (consumption)
04.A00.03.1490	02.12.2022	Bug correction. Watchdog reset 5 sec after Power On Reset. Added W-Dog indication on LED (Flashing Blue sky flash).
04.A00.04.1499 NEVER USED IN PRODUCTION	28.04.2023	"Battery LOW" information came too fast. The same measuring algorithm as the one for the other TXs has been applied.
04.A00.05.1502	02.06.2023	Supervision Interval fixed to '0' as default value. "User WR" access set for the 'Battery Threshold' parameter.