

# TOUCH 220

Software version: 1.17.4

Release date: 14/09/2021

Hardware compatibility: All existing hardware

Article numbers: NWATES0100

## ENHANCEMENTS & NEW FEATURES

Reference	Description
0003678	Added the option 'SIP call' to the output types to alert that a SIP call is incoming.
0003572	Check every 30 minutes if the SIP registration of the device is successful.
0003482 / 0003481	New GUI design of nurse desk overview screen.
0003452	Added 'Code Blue' as input/output type.
0003451	Improved input activation. The inputs were edge-triggered before and are now level-triggered.
0003420	Added LB types 387 / 388.
0003439	Header 19 response will only be send back to the IP requesting the information instead of 255.255.255.255.
0003449	The buzzer setting for a specific call type will be ignored if the show setting of that type is disabled.
-	Improvements on the use of the asterisk ('*') in room numbers.

## BUGFIXES

Reference	Description
0003380	MSG10 and MSG11 fields are empty when sending medallion calls (of a regular room) to the Netrix.
0003397	Code blue state of device could be changed to toilet state by activating an (Touch) input configured as toilet.
0003450	Show info 1-5 did not filter the nurse desk list.
0003381	Numerical Code Blue type to Netrix (MSG7) was 1 instead of 0.