EN Safety instructions

- The installation location must be within the reception range of the Carephone.
- The SP-20M must be installed indoors in order to protect it from the weather. It must be fixed on the frame of a door or a window, and the magnet should be installed on the moving part.
- When using the SP-20M for activity monitoring purposes, ensure that the door is used on a daily basis.
- Always carry out a wireless transmitter range test after replacing the battery. Please refer to the Carephone user manual.

! Installation and initial operation should only be carried out by trained service personnel.

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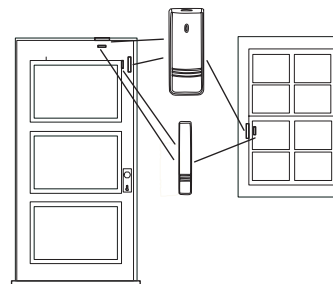
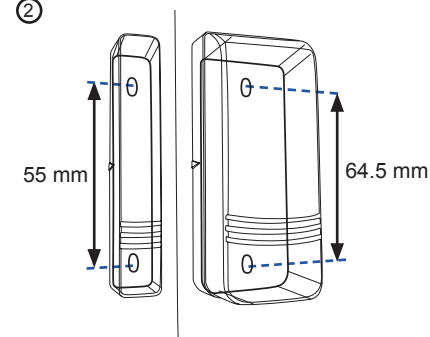
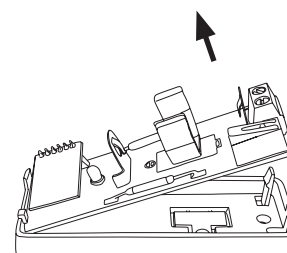
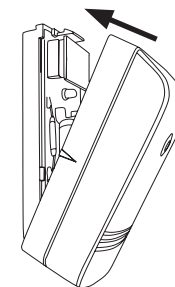
TeleAlarm Europe GmbH
Hertzstraße 2
04329 Leipzig
Germany
www.telealarm.com

EN Operating Instructions

The SP-20M is intended to be used in conjunction with activity monitoring operations managed by TeleAlarm Carephones or for transmission of specific events.

When used for activity monitoring, the Carephone can initiate a call for help automatically if there is no sign of activity.

The SP-20M can be activated by the movement of a magnet, or by the opening of an electrical contact. Terminal block contacts, normally closed.

When the top of the housing is removed from its base, a tamper switch detects this event, which is then transmitted to the Carephone.

Initial Operation

The SP-20M has its own individual code, which must be registered to the Carephone.

Put the Carephone into wireless transmitters registration mode in accordance with the Carephone user manual instructions.

After inserting the battery into the SP-20M battery holder, press and release the tamper switch to transmit the SP-20M code to the Carephone.

Installation

1. Select the installation location. ①
2. Remove the top of the housing from its base and remove the electronic circuit, using standard ESD precautions. ③
3. Fix the housing base in the desired location and reinsert the electronic circuit. ②
4. Install the magnet. The magnet can be fitted on either long side of the SP-20M.
5. Insert the battery on the battery holder.
6. Register the SP-20M code into the Carephone.
7. Fit the top of the housing onto its base. ④

The Carephone will be notified if the magnet is detached from the detector. The distance between the SP-20M and magnet must be ≤ 13 mm.

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